

Silverlon® Guidelines for Use

Silverlon® Roll With Integra® or STSG



1. For Silverlon® to be effective and non-adhering, the fabric must remain moist. Moisten the Silverlon® and cover gauze in sterile water prior to application.

(If microbial contamination is suspected on the wound bed, initiate Silverlon® 2 to 3 days prior to Grafting).



3. Cover the Silverlon® with sterile water moistened gauze and Kerlix.



5. Check every 4 to 6 hours to ensure the dressings remain moist. Remoisten every 8-12 hours with Sterile Water. Saturation of gauze is not necessary.



7. Replace outer gauze daily or PRN and moisten with sterile water.



2. Wrap wound with the moistened Silverlon® roll. Roll must be in direct contact with wound. Overlap Silverlon® to cover all wound surfaces.



4. Hold dressing in place with tubular stretch net.



6. The Silverlon® dressing can be periodically removed to visually inspect the wound for hematomas or purulent drainage. If excess purulent drainage is present, the Silverlon® dressing can be rinsed with water to remove excess exudate and reapplied to the wound surface.



8. Moisten Silverlon® dressings as needed prior to removal. Do not forcefully remove the dressing. Silverlon® should be replaced every 3-7 days.

Caution: When removing the dressing, if it is dry and sticking to the wound bed, use warm Sterile Water moistened gauze compresses to re-hydrate until the dressing can be gently removed, starting at the corners of the dressing.