



www.Silverlon.com

Silverlon® Surgical Post-Op Island Dressings

Better Technology. Better Outcomes.™

Patient Discharge Instructions

- **The Silverlon dressing should be left in place and worn continually for 7 days.**
- **If the physician's orders allow the patient to shower, the dressing is to be left on at all time. It is ok to get it wet.**
- **If the dressing adhesive boarder begins to peel away from the patient's skin, the patient may add addition surgical tape to hold the dressing in place.**
- **The dressing will be removed at the physician office on the first post-op follow up visit or at the professional discretion of the clinical staff.**

Note: Nursing home patient discharge.

It is recommended discharging a nursing home resident with an unopened Silverlon Surgical Island Dressing for the intake nurse to put on after she inspects the incision.