



## Silverlon® Surgical Post-Op Island Dressings

Better Technology. Better Outcomes.™

### OR Guidelines for Use

Silverlon® Surgical Post-Op Island Dressings are antimicrobial barrier wound dressings that are placed in primary contact with the surgical incision and may be used for up to 7 days.

### Applying the Silverlon® Island Dressing in the OR at Closure

- 1) *Cleanse* the incision using standard protocols.
- 2) *Activate* Silverlon® by wetting the entire silver surface with sterile water

#### Wetting tips

Remove dressing from pouch and fold back the release liner exposing the Silverlon® pad keeping the adhesive tape borders covered on the sides.

Using a bulb or piston syringe, or a bottle of sterile water, evenly wet the entire silver pad surface with 1 to 2 ounces of sterile water. Evenly pour the water across the entire surface of the Silverlon® pad ensuring moisture level throughout the pad.

- 3) *Remove* the release liner from the adhesive tape.
- 4) *Center and secure* the Silverlon® Island Dressing (silver surface down) directly over the surgical incision and onto peri-wound skin. Gently press down the tape borders to ensure good skin contact.

### General Precautions

- For external application only, remove prior to MRI scanning.
- Silverlon® dressings are contraindicated for patients with known sensitivity to silver or nylon.
- *Silverlon® Surgical Dressings have been reported to reduce the pain, swelling and inflammation from acute wounds.*

### Silverlon® Surgical Post-Op Island Dressing Sizes

Dressing Name	Tape Size	Pad Size	Product #	Unit Sale
Silverlon® Island Dressing	3" x 4"	2" x 2.5"	ID-34	10
Silverlon® Island Dressing	4" x 4"	2" x 2"	ID-44	5
Silverlon® Island Dressing	4" x 6"	2" x 4"	ID-46	5
Silverlon® Island Dressing	4" x 10"	2" x 8"	ID-410	5
Silverlon® Island Dressing	4" x 12"	2" x 10"	ID-412	5
Silverlon® Island Dressing	4" x 14"	2" x 12"	ID-414	5
Silverlon® Island Dressing	6" x 6"	4" x 4"	ID-66	5
Silverlon® WCD (Wound Contact Dressing) for cavity or circumferential wounds				
Silverlon® NPD (Negative Pressure Dressing) for use with NPWT (VAC™)				

[www.Silverlon.com](http://www.Silverlon.com)

1-888-551-0188