



Silverlon® Surgical Post-Op Wound Pad Dressings

Better Technology. Better Outcomes.™

OR Guidelines for Use

Silverlon® Surgical Post-Op Wound Pad Dressings are antimicrobial barrier wound dressings that are placed in primary contact with the surgical incision and may be used for up to 7 days.

Applying the Silverlon® Wound Pad Dressing in the OR at Closure

- 1) *Cleanse* the incision using standard protocols.
- 2) *Activate* Silverlon® by wetting the silver surface with sterile water.

Wetting tips

Remove dressing from pouch and let soak in bowl of sterile water prior to closure, wringing out excess water before application;

or

Open pouch at one end and add 1 to 2 ounces of sterile water to the dressing letting it lie until ready for use, and gently wring out excess water before application.

- 3) *Center and secure* the Silverlon® Surgical Dressing (silver surface down) directly over the surgical incision and onto peri-wound skin.
- 4) *Cover* the Silverlon® Dressing with Telfa® gauze, or other absorptive dressing of choice, and secure with tape.

General Precautions

- For external application only, remove prior to MRI scanning.
- Silverlon® dressings are contraindicated for patients with known sensitivity to silver or nylon.
- *Silverlon® Surgical Dressings have been reported to reduce the pain, swelling and inflammation from acute wounds*

Silverlon® Surgical Post-Op Wound Pad Dressing Sizes

Dressing Name	Size	Product #	Unit Sale
Silverlon® Pad Dressing	2" x 2"	WPD-22	5
Silverlon® Pad Dressing	2" x 3"	WPD-23	5
Silverlon® Pad Dressing	2" x 6"	WPD-26	5
Silverlon® Pad Dressing	2" x 8"	WPD-28	5
Silverlon® Pad Dressing	2" x 10"	WPD-210	5
Silverlon® Pad Dressing	2" x 12"	WPD-212	5
Silverlon® Pad Dressing	3" x 8"	WPD-38	5
Silverlon® Pad Dressing	3" x 10"	WPD-310	5
Silverlon® Pad Dressing	3" x 16"	WPD-316	5
Silverlon® Pad Dressing	4" x 4"	WPD-44	5
External Pin Sites/IV/CD 1" disk	1.5, 4.0, 7.0 mm	IVCD-X	10
Silverlon® WCD (Wound Contact Dressing) for cavity or circumferential wounds			
Silverlon® NPD (Negative Pressure Dressing) for use with NPWT (VAC™)			