



## Silverlon® Surgical Post-Op Wound Pad Dressings

Better Technology. Better Outcomes.™

### OR Guidelines for Use

*Silverlon® Surgical Post-Op Wound Pad Dressings are antimicrobial barrier wound dressings that are placed in primary contact with the surgical incision and may be used for up to 7 days.*

### Applying the Silverlon Wound Pad Dressing in the OR at Closure

- Cleanse the incision using standard protocols.
- Activate Silverlon® by wetting the silver surface with sterile water or saline.
- Center and secure the Silverlon® Surgical Dressing (silver surface down) directly over the surgical incision and onto peri-wound skin.
- Cover the Silverlon Dressing with pre-moistened gauze, and cover with absorptive dressing of choice, and secure with tape.

### Wetting tips: Silverlon Wound Pad Dressing

Remove dressing from pouch and let soak in bowl of sterile water or saline prior to closure, wringing out excess water before application;

or,

Open pouch at one end and add 15 to 45 mL of sterile water or saline to the dressing letting it lie until ready for use, wringing out excess water before application.

### General Precautions

- For external application only.
- Silverlon® dressings are contraindicated for patients with known sensitivity to silver or nylon.
- *Silverlon® Surgical Dressings have been reported to reduce the pain, swelling and inflammation from acute wounds*

### Silverlon® Surgical Post-Op Wound Pad Dressing Sizes

Dressing Name	Size	Product #	Unit Sale
Silverlon® Pad Dressing	2" x 2"	WPD-22	5
Silverlon® Pad Dressing	2" x 3"	WPD-23	5
Silverlon® Pad Dressing	2" x 6"	WPD-26	5
Silverlon® Pad Dressing	2" x 8"	WPD-28	5
Silverlon® Pad Dressing	2" x 10"	WPD-210	5
Silverlon® Pad Dressing	2" x 12"	WPD-212	5
Silverlon® Pad Dressing	3" x 8"	WPD-38	5
Silverlon® Pad Dressing	3" x 10"	WPD-310	5
Silverlon® Pad Dressing	3" x 16"	WPD-316	5
Silverlon® Pad Dressing	4" x 4"	WPD-44	5
External Pin Sites/IV/CD 1" disk	1.5, 4.0, 7.0 mm	IVCD-X	10
Silverlon® WCD (Wound Contact Dressing) for cavity or circumferential wounds			
Silverlon® NPD (Negative Pressure Dressing) for use with NPWT (VAC™)			