

Order Easy Ag<sup>®</sup>/Silverlon<sup>®</sup> Wound Care Products Online:

| [Adhesive Strips & Bandages](#) | [Wound Contact Dressings](#) | [Tubular Digit Sleeves](#) | [Breast Pad](#) | [Silverlon Home Page](#) |

## *Easy Ag<sup>®</sup> Antimicrobial Tubular Digit Sleeves*

**SAFE. STRONG. SIMPLE.**

DIGITAL SLEEVE DRESSING

SILVERLON<sup>®</sup>

silverlon.com (888) 551 0188

### **Directions for Home Use**

#### **How To Apply**



1. Easy Ag<sup>®</sup> Antimicrobial Tubular Digit Sleeve Bandages are ideal for application to wounds and burns located on the fingers and toes. This versatile tubular bandage can be applied to wounds, burns, blisters, and bites.
2. Cut to full length of digit as appropriate to cover site and apply.
3. Wet Dressing with Water.
4. Dressing may be removed, rinsed and reapplied again for up to 7 days.

**Caution: DO NOT allow the Dressing to restrict circulation.**

***Dressing effectiveness decreases if used with products containing petrolatum, iodine, or hydrogen peroxide.***



***Easy Ag<sup>®</sup> Wound Care Dressings  
General Directions for Home Use***

#### **Supplies Needed**

1. **Easy Ag<sup>®</sup> Wound Care dressings** should be sized to fit into the wound bed and up onto ½" (one-half inch) of normal skin around the wound. (The Easy Ag<sup>®</sup> dressing should last for up to 7 days if properly cared for as described below).
2. A **secondary cover dressing of choice** to be placed over the Easy Ag<sup>®</sup> dressing. Use tape, self stick gauze, tubular stretch knit or compressive wrap dressings to hold the Easy Ag<sup>®</sup> dressing in place.
3. **Clean water** for daily rinsing of Easy Ag<sup>®</sup> Dressing.

**Optional:** For heavily draining wounds, wound cleansers may be considered, such as Shur -Clens<sup>®</sup>, Saf – Clens<sup>®</sup>, Curasol<sup>®</sup>, Dermagran<sup>®</sup>, or Biolex<sup>®</sup>.

## General Daily Care

**Your goal is to keep your wound and the Easy Ag<sup>®</sup> dressing clean, moist, and covered. Recommend washing hands with warm water and soap before and after dressing change.** The Easy Ag<sup>®</sup> Dressing may be worn while showering unless otherwise directed by your physician or nurse. Please check with physician or nurse for exact showering directions. Soaking in a bathtub is not recommended.

## General Considerations During Use

1. Daily soaking and spraying with wound cleansers (optional, as listed above) may help remove wound debris and extend the life of the Easy Ag<sup>®</sup> dressing.

## For Draining Wounds

1. Remove and dispose of the outer absorbent cover dressings.
2. Gently remove Easy Ag<sup>®</sup> Dressing, **if any sticking occurs see warning instructions below.**
3. Rinse Easy Ag<sup>®</sup> Dressing thoroughly with clean water being sure to rinse away any exudate that maybe on the dressing (You may stretch the dressing by its corners to open up its weave to help loosen the exudate from the cloth), gently squeeze out excess water, and reapply to wound while moist.
4. Apply fresh cover dressings as per instructed by your physician or nurse.

**For Dry Wounds-** See [Professional Use Instructions and Protocols](#)

## ***Important to Remember***

- Remove Easy Ag<sup>®</sup> Dressing prior to MRI procedures.
- Do **not** remove the Easy Ag<sup>®</sup> Wound Care Dressing while dry or if Easy Ag<sup>®</sup> Dressing is sticking. **If sticking occurs**, rewet and saturate the Easy Ag<sup>®</sup> dressing waiting until it can easily be removed. If problems occur, including swelling, severe redness or sharply increased pain, contact your physician.

[Back to Tubular Digit Sleeve Bandage Purchasing Page](#)

---



For a printer friendly version of this Easy Ag<sup>®</sup>  
Tubular  
Wound Dressing Instructions Page, [please click here](#)



[Back to the Silverlon<sup>®</sup> Home Page](#)

---

## **Contact Information**

(Toll Free) Phone:  
1-888-551-0188

E-Mail Us at:  
[contact@silverlon.com](mailto:contact@silverlon.com)

FAX #:  
630-232-8005

---



All of the Easy Ag<sup>®</sup> and Silverlon<sup>®</sup>  
Wound Care, Burn Care, and Surgical  
Products listed on this website are **FDA  
Cleared.**



Copyright © 1998-2011 Argentum  
Medical, LLC.  
All Rights Reserved.

Proudly  
Manufactured



in the USA